


ENDOTINE
triple

The low profile ENDOTINE Triple™ fixation device provides high performance results

Low Profile Three tines on an ultra thin platform designed to reduce palpability and visibility.

High Performance Predictable brow elevation and secure tissue fixation provided by our proven MultiPoint Technology (MPT™).

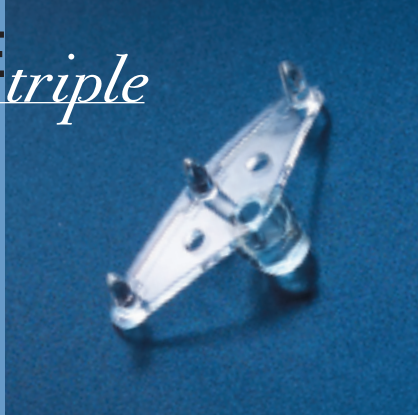

An Expanded Approach to Forehead Lifts

The ENDOTINE Triple features only three tines versus five for the ENDOTINE Forehead™, and a platform that is 60% smaller.



Coapt

Fewer Tines— More Patients

The Triple provides the security and predictability of ENDOTINE® multipoint fixation to patients who are not currently candidates because of concerns with palpability, visibility, sensitivity or prolonged absorption.

Sensitive patients Three 3.0mm tines and an ultra thin platform offer security without sensitivity for patients who may be affected by palpability.

Balding males or those with receding hairlines. With a low profile for reduced visibility and a petite size for a sub-10mm incision, the Triple is ideally suited for patients who are especially concerned with noticeable scarring or swelling.

High hairlines Reduced visibility and incision size allow the Triple to be placed confidently below the hairline.


Busy, active patients With one third the mass of an ENDOTINE Forehead, the Triple will more quickly absorb to an impalpable size, satisfying sophisticated and discriminating patients who demand a rapid recovery and quick return to daily activities.

the ENDOTINE TRIPLE PROCEDURE


1 MAKE THE INCISION




2 MAKE THE HOLE

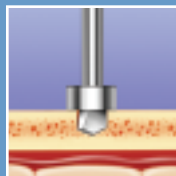

3 PLACE THE DEVICE

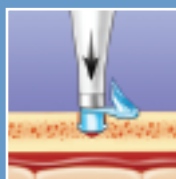

4 WITHDRAW THE TOOL

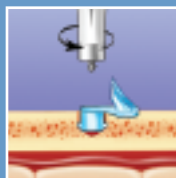

5 SECURE THE TISSUE



CAT#	PRODUCT NAME	PRODUCT DESCRIPTION	QUANTITY
CFD-010-0167	ENDOTINE Triple™	Bioabsorbable brow fixation device with 3.0mm tine height, including disposable drill bit and insertion tool	1 each
CFD-899-4000	Coapt Manual Surgical Drill	Manual surgical hand drill for use with ENDOTINE implants	1 each

PATENT PENDING. THIS PRODUCT IS COVERED BY ONE OR BOTH OF THE FOLLOWING U.S. PATENTS: 6,645,226 AND 6,485,503.



Coapt Systems, Inc.
1820 Embarcadero Road
Palo Alto, CA 94303
800.963.7670 Toll Free
650.213.9377 Fax
www.coaptsystems.com

©2005 Coapt Systems, Inc. All rights reserved. Coapt and ENDOTINE are registered trademarks of Coapt Systems, Inc. ENDOTINE Triple, ENDOTINE Forehead and MPT are trademarks of Coapt Systems, Inc.

LBL-010-7046 Rev B